IEA Medical Augenmaske

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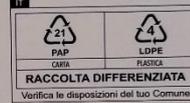
EN - APPLICATION: The eye masks are suitable for cold and heat treatment. They have a pain-relieving effect on complaints such as swollen eyes, eye pain, headaches, migraines and fever. **APPLICATION OF COLD:** For a cold compress, place the eye mask in a protective cover in the freezer for approx. 30 minutes. For optimum effect, use the eye mask at short intervals for a maximum of 20 minutes. **HEAT APPLICATION:** For a heat compress, heat the mask in the microwave for about 15 seconds at 600 watts. Then knead well to distribute the heat. **REUSABLE:** The eye masks can be used several times and are durable and resilient. The material is extremely hygienic for the skin. It retains heat and cold for a long time. **SAFETY:** Watch the eye mask as it heats up. If the face mask expands, switch off the microwave immediately. The beads of the mask must not be swallowed! Keep out of the reach of children.

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INTCO Medical (HK) Co., LTD
 Flat/RM 19C, Lockhart Centre,
 301-307 Lockhart Road
 Wan Chai, Hong Kong

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omplaints such as swollen eyes, eye pain, headaches, r for approx. 30 minutes. For optimum effect, use the e microwave for about 15 seconds at 600 watts. Then e material is extremely hygienic for the skin. It retains ave immediately. The beads of the mask must not be

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EC REP MedNet EC-REP GmbH
IEA International Trading GmbH