

IEA Medical Fieberthermometer

Anwendung:


- Vorbereitung: Thermometer einschalten, abwarten, bis LCD-Anzeige bereit ist.
- Messung: Unter Zunge oder Achsel platzieren, ruhig halten. Beep-Signal zeigt Messende.
- Nach Gebrauch: Thermometer reinigen und ausschalten.

Warnhinweise:

- Nicht für andere Messorte verwenden.
- Nicht selbst reparieren oder teilen.
- Kinder nur unter Aufsicht anwenden lassen.
- Batterien sicher entsorgen und nicht verschlucken lassen.
- Gerät nicht hohen Temperaturen oder Chemikalien aussetzen.

Kontakt-Hersteller:

- market3@genial.cn

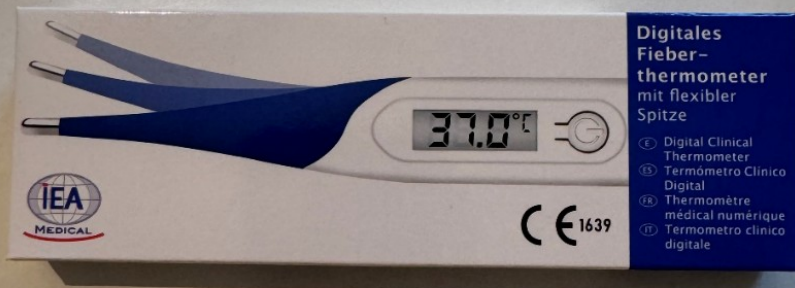

IEA MEDICAL

Digitales Fieberthermometer mit flexibler Spitze

Messskala 32°C - 42,9°C
 Range/Escala de medición/Échelle de mesure/Scala di misura

Messung unter 60s
 Measurement under/Mediación por debajo de/
 Mesure en moinsde/Misurazione sotto

Messmethoden Oral, Rektal
 Measurement methods/Métodos de medición/Axillary Méthodesde mesure/Metodidimisurazione

Bitte lesen Sie vordem Gebrauch die Gebrauchsanweisung!
 Please read the instructions for use before use!
 Lea las instrucciones de uso antes de utilizarlo!
 Veuillez lire les instructions d'utilisation avant de l'utiliser!
 Leggere le istruzioni per l'uso prima dell'uso!

CE 1639

MD

Recycling symbols: PAPER (PAP), CARTRIDGE (CARTON), and others.

Barcode: 4 260764 170661

LOT T230819 2023.8.17



Digitales Fieberthermometer mit flexibler Spitze

CE 1639

MD

Recycling symbols: PAPER (PAP), CARTRIDGE (CARTON), and others.

Barcode: 4 260764 170661

LOT T230819 2023.8.17

Digitales Fieberthermometer mit flexibler Spitze

Medipac GmbH
 W.-v.-d.-Röhe-Strasse 8 - 88074 München

CHANGKONGZHENALTECHNOLOGY CO., LTD.
 1-6-021118 Road, Anyue, Changping District,
 Cooperation Zone, Zhenjiang Town, Jiangsu
 Zhenjiang City, Jiangsu Province, 212637, P.R.C.

EMERGENCY TECHNICAL SUPPORT
 Contact: +86-512-5615471
 Tel: +86-88-4570531-0
 Email: hfr@iea-medical.com